



# Good study results, better night's sleep

## Alice PDx Diagnostic System

The Alice PDx is a portable, diagnostic recording device for OSA screening, follow-up and diagnostic assessment of cardio-respiratory sleep disorders. The Alice PDx enables you to test your patients outside of the lab without compromising study results, helping you to avoid the costs associated with retesting.

The Alice PDx incorporates our unique Good Study Indicator (GSI).\* The GSI is predicated on airflow and oximeter signal quality and displays the amount of "good quality data" needed for a study to be

complete and valid. The indicator measures patient airflow, gathered by the nasal cannula and/or the oral thermistor, airflow from therapy devices, and pulse oximetry, gathered by the SpO<sub>2</sub> sensor. Without either of these signals, the sleep study would be declared diagnostically invalid because of insufficient data.



### Optional equipment

ECG yoke; ExG yoke; ECG and neurological electrodes (for ExG); therapy device communication cable; ventilation device communication cable

### Basic PC recommendations

Windows®-compatible personal computer running Windows Vista Business, Windows 7 professional (64 bit), Windows 8 professional (64 bit) with a CD-RW drive.

- Intel® Pentium® 3 processor (500 MHz) or greater, processor (1 GHz) required for Windows Vista computers
- 512 MB system memory, 1 GB or higher for Windows Vista computers

### The base configuration channel set supports testing requirements for levels three and four and includes:

- nasal pressure cannula
- oral thermistor
- zRIP abdominal and chest belts
- SpO<sub>2</sub>
- body position

In addition to the basic channel set indicators, there are also indicators for the optional ECG and ExG yokes and the optional therapy interface.

#### Battery Indicator

indicates power level.

#### Memory Card Indicator

provides estimated memory capacity.

#### Good Study Indicator


configurable for the number of hours required for a valid study

#### Device Error Indicator

indicates the need for service.

#### Status Indicator

displays green to indicate proper operation or yellow for operation error (i.e., disconnected sensor).

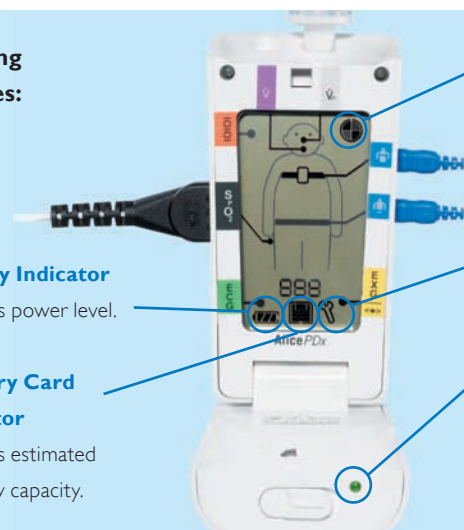

# Product specifications

## Specifications:

Total channels:	Up to 21 with Optional ECG and ExG Yokes
Base channels:	10 Channels: Pressure-Based Flow (w/snore) and Thermal Airflow, zRIP Effort (2), Body Position, SpO <sub>2</sub> (also Pleth and Pulse Rate) and Patient Marker. Up to eight parameters from Philips Respironics therapy devices may be acquired, including pressure, flow and leak.
Optional yokes/ channels:	ECG Yoke; 7 channels: 3-lead ECG providing 3 measured and 4 derived channels ExG Yoke; 7 channels: 4 Neuro Channels (EEG or EOG) and 3 Differential EMG, plus references and ground
Input impedance:	10 MΩhm per elec. 20 MΩhm differential 2 MΩhm per elec. 4 MΩhm differential 2 MΩhm per elec. 4 MΩhm differential
Bandwidth:	ECG: 0.318 Hz to 81 Hz EEG: 0.318 Hz to 35 Hz EMG: 9.7 Hz to 86 Hz
Input signal range:	ECG: +/- 4 mV EEG: +/- 500 uV EEG: +/- 150 uV
Digital resolution:	Up to 16 bits
Initial sample rate:	Up to 1,000 Hz
Max storage rate:	200 Hz for ECG/ExG, 100 Hz for Effort and Thermal Flow, Pressure Flow 200 Hz, Snore 500 Hz, Body Pos. 1 Hz
Communication interfaces:	Serial Protocol USB PC cable, SleepLink and Serial communication cables for Philips Respironics Therapy devices

## System physical characteristics:

Alice PDx unit:	
Size:	5" L x 3" W x 2" H (12.7 cm x 7.62 cm x 5.08 cm)
Weight:	Approximately 8 oz (230 grams), (weight does not include batteries)
System power:	Three AA (1.5V) alkaline batteries, 0.43 watts
Requirements:	Alice PDx can support the use of rechargeable batteries

## Ordering information:

Item No.	Description
1043844	Alice PDx Device Kit, International
<b>Optional components:</b>	
1048415	ExG Yoke, Alice PDx, International
1040810	ECG Yoke, Alice PDx, International
<b>Accessories:</b>	
1040806	Alice PDx to SleepLink Cable
1040805	Alice PDx to RS-232 Sleep Therapy Cable
1116888	3 foot SpO <sub>2</sub> extension cable
936	SpO <sub>2</sub> Reusable Sensor, Adult Finger Clip
953	SpO <sub>2</sub> Reusable Flex Sensor, Adult
954A	Adult Reusable Sensor Wrap (25/pack)
P1328-60	Pro-Flow Nasal Canula, Adult (60/pack)
P1379	Oral Thermistor, Alice PDx
P1388	Nasal-Oral Thermistor, Alice PDx
P1391	CPAP Titration Kit (for non-Philips Respironics therapy devices)
<b>Software:</b>	
	Sleepware (included with purchase)

Philips Respironics and Alice are trademarks of Koninklijke Philips N.V., Inc. and its affiliates. All rights reserved.

©2015 Koninklijke Philips N.V. All rights are reserved. Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

Broudy ASG 1/30/15 MCI 4105798 PN 1057476

Philips Respironics Asia Pacific  
+65 6882 5282

Philips Respironics Australia  
+61 (2) 9947 0440

Philips Respironics United Kingdom  
+44 800 1300 845

Philips Respironics  
1010 Murry Ridge Lane  
Murrysville, PA 15668 USA  
Customer Service  
+1 724 387 4000

[www.philips.com/respironics](http://www.philips.com/respironics)

